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USE AND MAINTENANCE

of reusable surgical instruments

Read carefully the instructions of this sheet. Wrong usage and/or inadequate procedures may cause an early deterioration and damage of the medical devices. Our surgical instruments must be used only by trained specialists.

FIRST USE

Each instrument must be properly cleaned and sterilized prior to first usage.

INSPECTION AND CONTROL

Carefully inspect the device prior to every usage in order to check its functionality and eventual damages. Refrain from using damaged instruments. Do not attempt for any reason to repair instruments, contact immediately our customer care and repair service.

STERILIZATION

Follow the instructions of the manufacturer of the sterilization devices and the procedures provided by your organization. We recommend to sterilize the devices in autoclave according to the following parameters:

MATERIAL	min	°C	BAR
Surgical instruments, surgical drapes, tempered glass	5-7	134	2,2
Rubber and plastic	15-20	121	2,1

Other sterilization means are under user's responsibility.

POST SURGICAL CLEANING / MAINTENANCE

Wash the instrument right after use. A late cleaning may cause the formation of residuals on the device that might jeopardize further sterilizations.

Decontamination

Immediately remove blood stains, saline solutions, tissues and other contaminant agents by a thorough washing with distilled/deionized water. Put all the devices in a disinfectant solution.

Cleaning and manual cleaning

Thoroughly clean the instruments with a non-corrosive detergent, better if enzymatic. Use delicate, non-abrasive brushes. Cannulated instruments: flush for at least 5 times with 10 ml of solution.

Ultrasound cleaning

Please refer to manufacturer's instructions.

1. Place the instruments inside the washer, for a maximum of 5 minutes, with the detergent solution prepared according to manufacturer's instructions.
2. Make sure that the instruments are completely in touch with the solution and that all the joints are well opened.
3. You may protect the most delicate parts with silicone caps. Make sure that such protections do not interfere with proper cleaning.
4. Avoid contacts among instruments while performing ultrasound washing

We strongly discourage use of ultrasound cleaning of diamond knives

Procedure with machine

Use a machine with the following features:

- approved efficient disinfection (i.e. CE marking)
 - the machine must have proper baskets capable of storing small delicate objects and with tubing connection for the washing
 - washing with distilled/deionized water
 - air used for drying must be filtered
1. Carefully store the products in the basket
 2. Connect the cannulated instruments to the proper connecting devices for a correct machine washing
 3. Start the program
 4. Double check all the devices

Rinsing

Thoroughly rinse the instruments with distilled/deionized water making sure to have the detergent completely removed from instruments' surface.

Drying

Dry the instruments with compressed air, in compliance with the procedures approved by your organization.

DO NOT EXPOSE THE INSTRUMENTS TO TEMPERATURES ABOVE 135°C.

For further info you can download a more detailed "use and maintenance" sheet at www.janach.it

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