CATARACT GLAUCOMA REFRACTIVE


EDI.12

we elaborate Concepts
to give a clearer Vision

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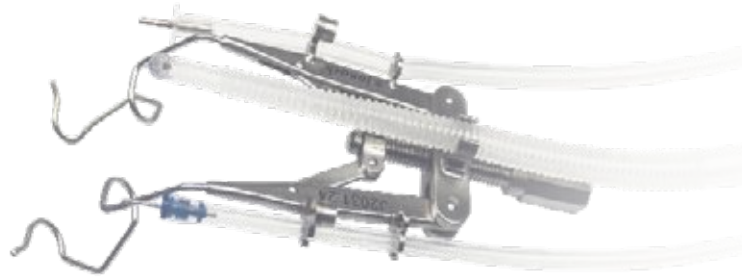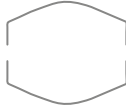
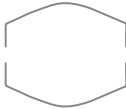
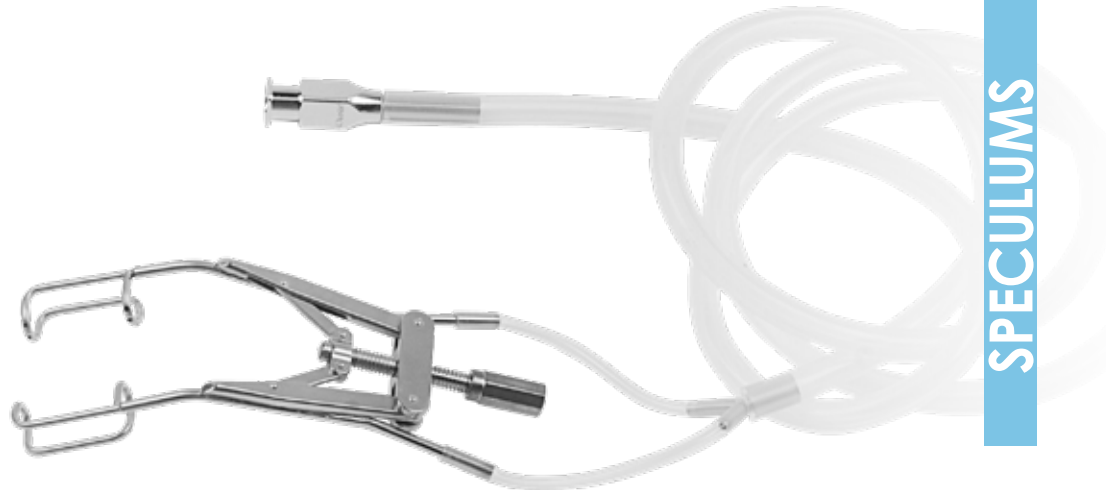
Adjustable speculum, reverse, **14 mm solid blades**, titanium designed by P. Vinciguerra M.D.

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
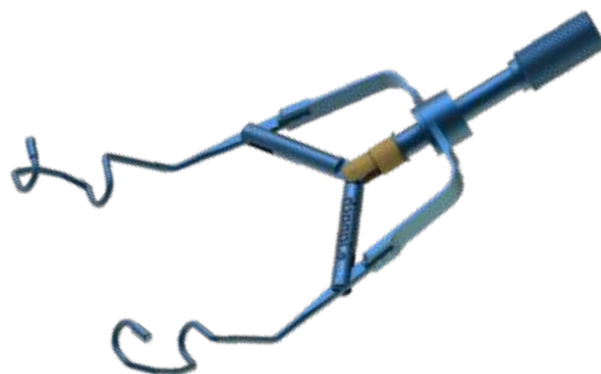
Reverse speculum with **15 mm closed blades**, titanium

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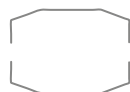J2049.1

Premature infant size,
wire speculum,
very small, 5 mm long blade

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J2049.2

Premature infant size,
wire speculum, with flanges,
5 mm long blade

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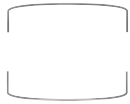
J2049.3


Infant size, wire speculum,
with flanges, 8 mm long blade

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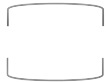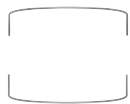
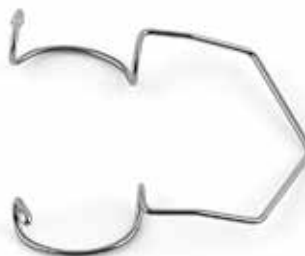
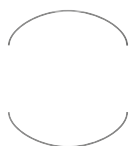
Speculum, light, **self-locking**, 14 mm long closed blade

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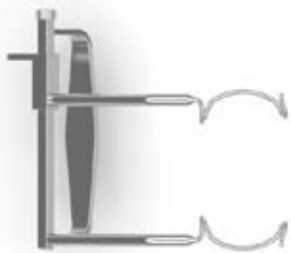
Speculum, light, **self-locking**, 16 mm long, open and convex blade

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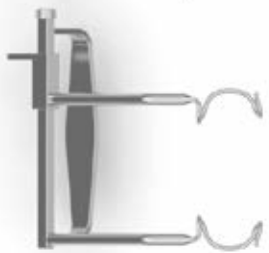
Self-locking speculum with alignment system of the valves by means of a support created in the temporal area. This support, helps to reduce the weight induced by the speculum on the bulb of the patient, thus limiting unwanted increments of the intraocular pressure. It facilitates the docking step in assisted surgeries making the procedure easier

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
Atraumatic, non-invasive, suggested for general use in all eyes designed by P. Crozafo® M.D.

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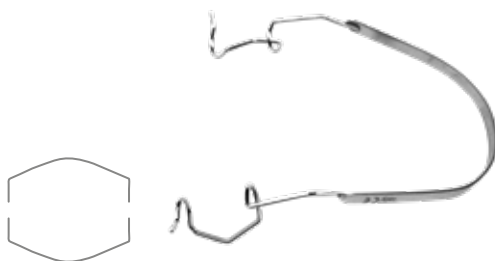
Wire speculum **15 mm long blade** with large frame. Recommended for diagnostic and surgical procedures. The wide frame allows easy placement and removal

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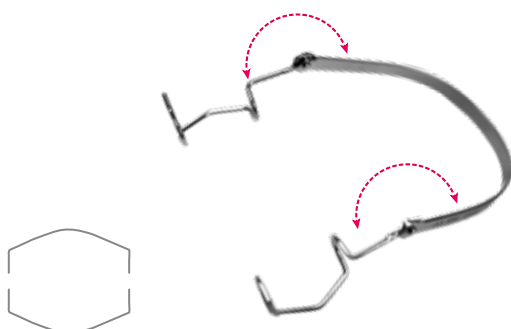
Reverse wire speculum 15 mm long blade with large frame. Recommended for diagnostic and surgical procedures. The wide frame allows easy placement and removal

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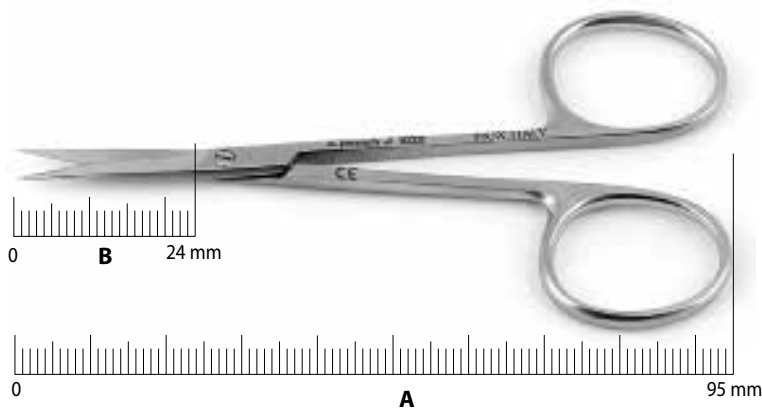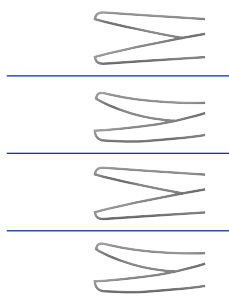
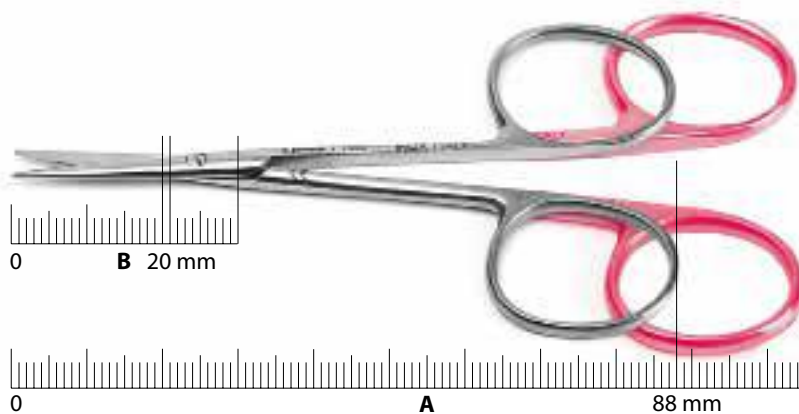
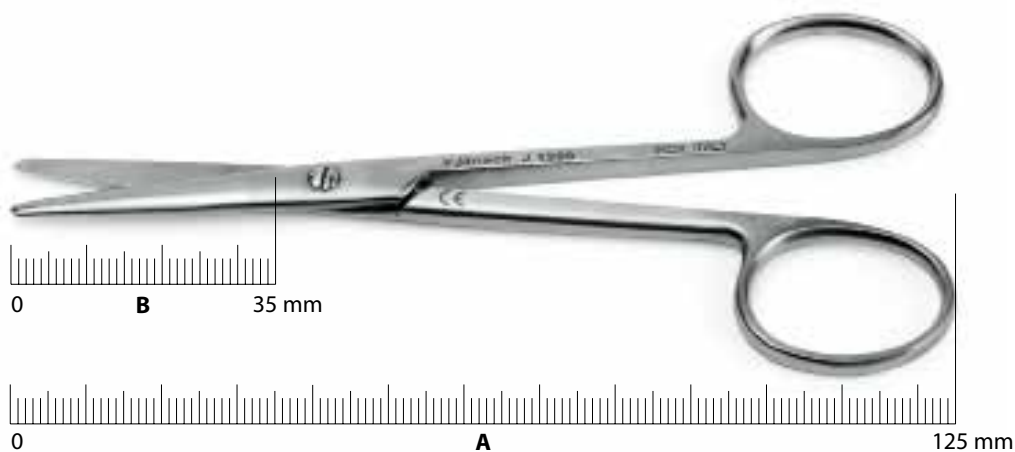
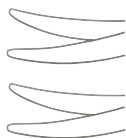
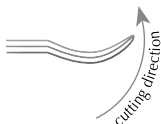
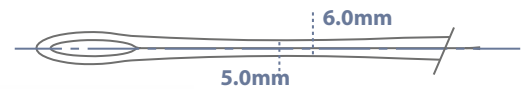
MICS CAPSULORRHEXIS FORCEPS CONTROLLED OPENING

J3135.1E titanium version EVOLine

Pinza capsuloressi per MICS con apertura controllata, branche **11 mm**, apice ovale per **incisione da 1.8 mm**
 MICS capsulorrhesis forceps with controlled opening, **11 mm long oval shaped tips**, for **1.8 mm incisions**
 designed by Giannetti M.D.

J3135.1

Pinza capsuloressi per MICS con apertura controllata, branche **11 mm**, apice ovale per **incisione da 1.8 mm**
 MICS capsulorrhesis forceps with controlled opening, **11 mm long oval shaped tips**, for **1.8 mm incisions**
 designed by Giannetti M.D.

J3141E titanium version EVOLine

Pinza capsuloressi per faco-microcoaxial con apertura controllata, **branche 12 mm**, per **incisione da 1.8 mm**
 Capsulorrhesis forceps for microcoaxial phaco with controlled opening, **12 mm long tips**, for **1.8 mm incisions**
 designed by P. Crozafon® M.D.

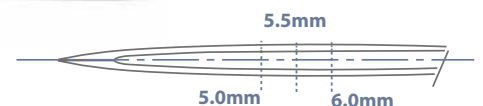
J3141

Pinza capsuloressi per faco-microcoaxial con apertura controllata, **branche 12 mm**, per **incisione da 1.8 mm**
 Capsulorrhesis forceps for microcoaxial phaco with controlled opening, **12 mm long tips**, for **1.8 mm incisions**
 designed by P. Crozafon® M.D.

J3135.4

Pinza capsuloressi MICS con apertura controllata, branche 11 mm punte sottili 90°, per **incisione da 2.2 mm**
 MICS capsulorrhesis forceps with controlled opening 11 mm long 90° fine tips, for **2.2 mm incisions**. designed By D. Surace M.D.

J3135.3

Pinza capsuloressi per MICS con apertura controllata, lunghezza branche 11 mm, apice ovale per **incisione da 2.2 mm**,
 manico piatto in titanio
 MICS capsulorrhesis forceps with controlled opening, 11 mm long oval shaped tips, for **2.2 mm incisions**, titanium flat handle.
 designed By Formenti M.D.

J3142.2 Giardini

Micropinza capsuloressi per MICS, **apertura max 2.0 mm**
 MICS capsulorrhesis forceps, **controlled opening 2.0 mm**

OVERLAPPING / COAXIAL CAPSULORRHEXIS FORCEPS

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
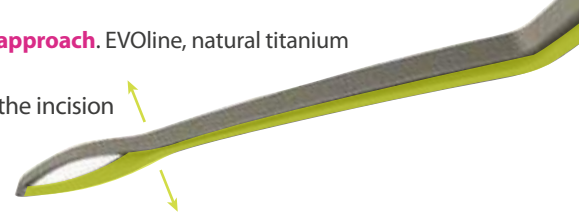
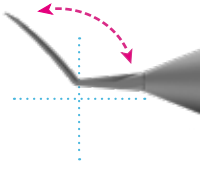
Overlapping system cross action micromechanism avoids stressing the incision

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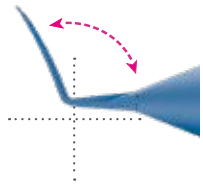
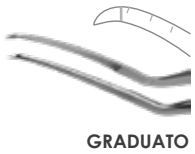
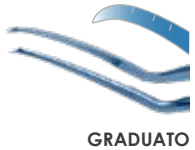
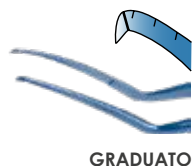
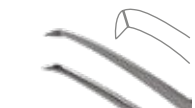
Capsulorrhesis forceps, very fine, angled, 11 mm, **graduated**, curved arms, sharp, very angled tips, ergonomic handle designed By Galan M.D.


J3133 Utrata

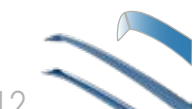Pinza per capsuloressi, angolata, dorso poco curvo da 11 mm, **apice acuto**, manico ergonomico
Capsulorrhesis forceps, angled, 11 mm, slightly curved arms, **sharp tips**, ergonomic handle


J3133E titanium version EVOLine

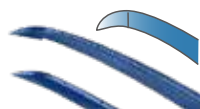Pinza per capsuloressi, angolata, dorso poco curvo da 11 mm, **apice acuto**, manico ergonomico
Capsulorrhesis forceps, angled, 11 mm, slightly curved arms, **sharp tips**, ergonomic handle

J3138 Corydon

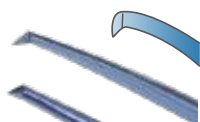Pinza per capsuloressi, leggera, angolata, dorso curvo da 12 mm, apice acuto, in titanio
Capsulorhexis forceps, lightweight, angled, 12 mm, slightly curved arms, sharp tips, titanium

J3139 Utrata

Pinza per capsuloressi, leggera, angolata, dorso curvo da 12 mm, apice acuto, molto angolato, in titanio
Capsulorhexis forceps, lightweight, angled, 12 mm, slightly curved arms, strongly angled, sharp tips, titanium

J3139.1

Pinza per capsuloressi, molto fine, angolata, dorso curvo da 11 mm, graduato, apice acuto, molto angolato, in titanio
Capsulorhexis forceps, very fine, angled, 11 mm, graduated, slightly curved arms, sharp, strongly angled tips, titanium designed By Galan M.D.

J3126.4 Inamura

Pinza per capsuloressi, manico piatto, branche da 11 mm, apice acuto, in titanio
Capsulorhexis forceps, flat handle, 11 mm sharp tips, titanium

FLAT HANDLE CAPSULORRHEXIS FORCEPS

J3126.1

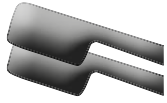
Pinza per capsuloressi, angolata, dorso curvo da 11 mm, presa con incavo, apice acuto
Capsulorhexis forceps, angled, 11 mm, slightly curved arms, sharp, notched tips designed By L. Buratto M.D.

J3129 Corydon

dorso curvo da 12 mm, graduato, apice acuto / 12 mm long tips, graduated, sharp tips

PRECHOPPERS AND NUCLEUS SPLITTERS

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Phaco chopper, 10 mm angle to tip wedge-shaped rounded tip, 70° cutting edge, L=90 mm, suggested for quick-chop
designed by Pezzola M.D.

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Nucleus manipulator, angled, 12 mm curved arm, with straight, **olive-shaped tip**
designed by L. Buratto M.D.

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Nucleus manipulator and splitter, angled, 10 mm angle to tip, paddle-shaped tip

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IOL manipulator, angled, 12 mm angle to tip
designed by Sinsky M.D.

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
IOL manipulator, short, delicate, angled, 12 mm angle to tip
designed by L. Buratto M.D.

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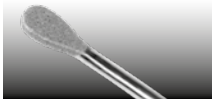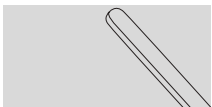
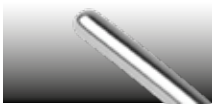
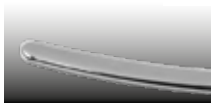
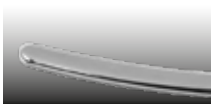
**J2400.2**

Spatola De Wecker per iride 1,5 mm
 IRIS spatula, 1.5 mm, De Wecker model

IOL MANIPULATOR PUSH-PULL / VITREOUS REMOVAL SPATULA

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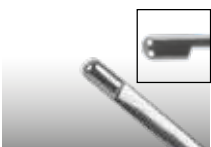
Spatula to remove vitreous residues in anterior chamber, 10 mm angle to tip, with notch on the distal end

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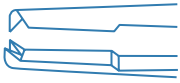
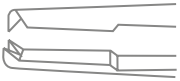
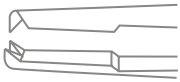
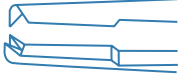
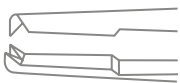
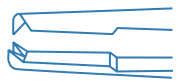
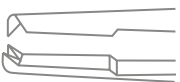
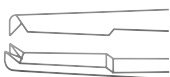
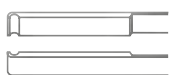
Corneal forceps, straight, 1x2 straight, 0.30 mm teeth, with tying platform

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Suturing forceps, straight, 1x2 oblique 0.12 mm teeth, with tying platform

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Conjunctival and fixation forceps, straight, 1x2 straight 0.40 mm teeth, with tying platform

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Conjunctival and fixation forceps, straight, 1x2 straight 0.70 mm teeth, with tying platform

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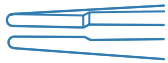
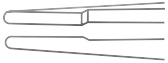
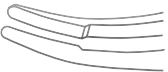
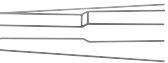
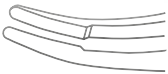
Curva / Curved

6-0 / 8-0

Consigliate per suture da 6-0 a 8-0 / Recommended for 6-0 to 8-0 sutures

J3914

Pinza serrafili, retta, con piani delicati da 5 mm, in titanio
Tying forceps, straight, with delicate 5 mm tying platform, titanium

8-0 / 10-0

J3917

Curva, con piani delicati da 6 mm, in titanio
Curved, with delicate 6 mm tying platform, titanium

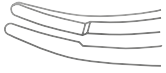
8-0 / 10-0

Consigliate per suture da 8-0 a 10-0 / Recommended for 8-0 to 10-0 sutures


J3207E Tennant, titanium version EVOLine

Pinza serrafile, curva, con piani da 7 mm / Tying forceps, curved, with 7 mm tying platform


J3207 Tennant

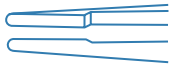Pinza serrafile, curva, con piani da 7 mm / Tying forceps, curved, with 7 mm tying platform

J3207.1E Tennant, titanium version EVOLine

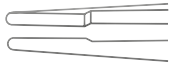Retta, con piani da 6 mm / Straight, with 6 mm tying platform


J3207.1 Tennant

Retta, con piani da 6 mm / Straight, with 6 mm tying platform

Consigliata per suture da 7-0 a 9-0 / Recommended for 7-0 to 9-0 sutures

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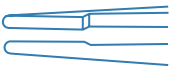
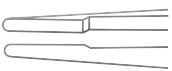
Tying forceps, straight, with 6 mm tying platforms, very delicate

Consigliate per suture da 8-0 a 11-0 / Recommended for 8-0 to 11-0 sutures

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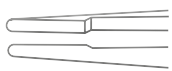Tying forceps, delicate, straight, blunt tips, with 6 mm tying platform

Consigliata per suture da 8-0 a 11-0 Recommended for 8-0 to 11-0 sutures

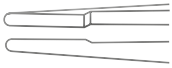
8-0 a 11-0


J3290

Pinza serrafile, retta, con piani delicati da 4 mm
Tying forceps, straight, with 4 mm tying platform, delicate

J3292

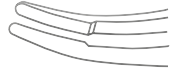curva / curved
Consigliata per suture da 8-0 a 10-0 / Recommended for 8-0 to 10-0 sutures

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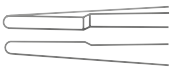
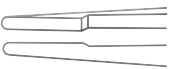
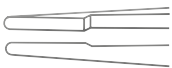
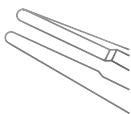
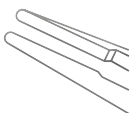
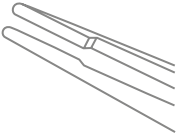
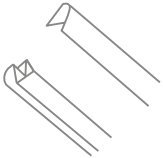
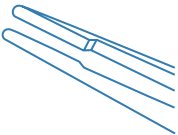
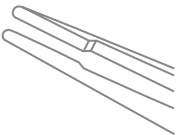
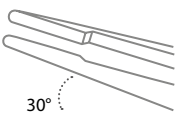
Tying forceps, angled, with 9 mm tying platform, titanium

J3909.2

con piani da 11 mm, in titanio / with 11 mm tying platform, titanium

Consigliata per suture da 8-0 a 10-0 / Recommended for 8-0 to 11-0 sutures

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Tying forceps, angled, with 8 mm tying platform, titanium

Consigliata per suture da 8-0 a 10-0 / Recommended for 8-0 to 10-0 sutures

8-0 / 10-0

JCP0008 SET PER SUTURA/SUTURING KIT

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set composto da:
includes:

pinza serrafili / tying forceps

bisturi 22.5° / 22.5° knife

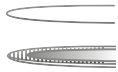
portaghi / needleholder

DRESSING FORCEPS / MOORFIELD


**J3200
Moorfield**

Pinza retta, delicata, zigrinata, apici acuti

Forceps, straight, delicate, serrated jaws, pointed tips

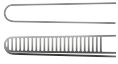
J3020

Pinza **anatomica**, delicata, retta, presa zigrinata da 0.60 mm

Dressing forceps, delicate, straight, 0.60 mm, serrated jaws

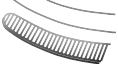
J3022

delicata, curva, presa zigrinata da 0.60 mm

delicate, curved, 0.60 mm, serrated jaws

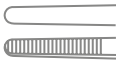
J3096.1

robusta, retta, presa zigrinata da 0.90 mm

strong, straight, 0.90 mm, serrated jaws

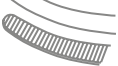J3096.2

robusta, curva, presa zigrinata da 0.90 mm

strong, curved, 0.90 mm, serrated jaws

MC. PHERSON/ NEEDLE HOLDER / HOLDER FORCEPS

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combines several functions in anterior segment surgery:
delicate tying forceps, IOL loading forceps and needle holder for microsutures

J3214.1E

Pinza **Mc. Pherson/needle holder**, cross action in titanio

Mc. Pherson/needle holder, cross action forceps, titanium

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Needle holder, very curved, very thin, without lock, overall length: 125 mm

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Stainless steel needle holder, curved, thin, without lock, overall length: 135 mm

Consigliati per suture da 10-0 a 11-0 / Recommended for 10-0 to 11-0 sutures

J4056 Barraquer

Port'aghi di Barraquer, a molla, punte rette / Barraquer needle holder, straight

NEEDLE HOLDERS

J4191

Port'aghi in acciaio, per sutura corneale, curvo, senza arresto, con piani delicati, da 103 mm
Stainless steel needle holder, curved, without lock, with delicate jaws, overall length: 103 mm

8-0 / 10-0

Consigliato per suture da 8-0 a 10-0 / Recommended for 8-0 to 10-0 sutures

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8-0 / 10-0

Consigliato per suture da 8-0 a 10-0 / Recommended for 8-0 to 10-0 sutures

J4195

Port'aghi in acciaio, per sutura corneale, curvo, senza arresto, con piani delicati, da 120 mm
Stainless steel needle holder, curved, without lock, with delicate jaws, overall length: 120 mm

8-0 / 10-0

Consigliato per suture da 8-0 a 10-0 / Recommended for 8-0 to 10-0 sutures

J3961

Port'aghi per sutura corneale, curvo, **senza arresto**, da 115 mm, in titanio
Needle holder, curved, **without lock**, overall length: 115 mm, titanium

9-0 / 10-0

Consigliato per suture da 9-0 a 10-0 / Recommended for 9-0 to 10-0 sutures

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J4124
retto, **con arresto** / straight, **with lock**

J4125
curvo, **con arresto** / curved, **with lock** 8-0 / 10-0

Consigliati per suture da 8-0 a 10-0 / Recommended for 8-0 to 10-0 sutures

J4131
Port'aghi in acciaio, curvo, senza arresto, da 135 mm
Stainless steel needle holder, curved, without lock, overall length: 135 mm 8-0 / 10-0

Consigliato per suture da 8-0 a 10-0 / Recommended for 8-0 to 10-0 sutures

Twister

J4197
Port'aghi Twister, curvo, **senza arresto**, da 128 mm, snodo anti-incarcerazione della sutura, punte delicate
SnagFree Twister needle holder, curved, **without lock**, delicate tips, overall length: 128 mm 10-0 / 11-0

Consigliato per suture da 10-0 a 11-0 / Recommended for 10-0 to 11-0 sutures

Twister

J4198
Port'aghi Twister, curvo, senza arresto, da 128 mm, snodo anti-incarcerazione della sutura, punte delicate
SnagFree Twister needle holder, curved, without lock, delicate tips, overall length: 128 mm 7-0 / 9-0

Consigliato per suture da 7-0 a 9-0 / Recommended for 7-0 to 9-0 sutures

Twister

J4199E
Port'aghi, lunghezza 125 mm, curvo, con sistema "Twister"(anti-incarcerazione della sutura), in titanio
SnagFree Twister needle holder, delicate tips, overall length: 125mm. 8-0 / 10-0
New handle with "Cross-action forceps style"

Consigliato per suture da 8-0 a 10-0 / Recommended for 8-0 to 10-0 sutures

designed by P.Vinciguerra M.D.

J4060

Port'aghi robusto, retto, **senza arresto**, da 145 mm
Needle holder, strong, straight, **without lock**, overall length: 145 mm

J4061

curvo, **senza arresto** / curved, **without lock**

J4064

retto, **con arresto** / straight, **with lock**

J4065

curvo, **con arresto** / curved, **with lock**

4-0 / 7-0

Consigliati per suture da 4-0 a 7-0 / Recommended for 4-0 to 7-0 sutures

J4080

Port'aghi in acciaio, retto, **senza arresto**, da 118 mm. Consigliato per suture da 6-0 a 8-0
Stainless steel needle holder, straight, **without lock**, overall length: 118 mm

J4081

curvo, **senza arresto** / curved, **without lock**

J4084

retto, **con arresto** / straight, **with lock**

J4085

curvo, **con arresto** / curved, **with lock**

6-0 / 8-0

Consigliati per suture da 6-0 a 8-0 / Recommended for 6-0 to 8-0 sutures

J4105

Port'aghi in acciaio, curvo, senza arresto, da 130 mm
Stainless steel needle holder, curved, without lock, overall length: 130 mm

8-0 / 10-0

Consigliato per suture da 8-0 a 10-0 / Recommended for 8-0 a 10-0 sutures

J4010

Port'aghi in acciaio per sutura palpebrale, **retto**, branche robuste, da 135 mm
Stainless steel needle holder, **straight**, strong jaws, overall length: 135 mm

J4011

curvo, branche robuste, da 135 mm / **curved**, strong jaws, overall length: 135 mm

5-0 / 6-0

Consigliati per suture da 5-0 a 6-0 / Recommended for 5-0 to 6-0 sutures

21G I/A bimanual CANNULAE

IRRIGATING

ASPIRATING

J2233.1

Cannula aspirazione curva da Ø 0.80 mm, punta sabbata, foro superiore Ø 0.35 mm, manico in titanio
Aspirating handpiece, Ø 21G, curved, sandblasted tip, 28G top port, titanium handle

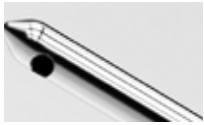
J2233.2

Cannula infusione curva da Ø 0.80 mm, a 2 fori laterali Ø 0.45 mm, manico in titanio
Irrigating handpiece, Ø 21G, curved tip, dual 26G side ports, titanium handle

J2233.3

Cannula aspirazione curva da Ø 0.80 mm, punta liscia, foro superiore Ø 0.35 mm, manico in titanio
Aspirating handpiece, Ø 21G, curved tip, 28G top port, titanium handle

J2233.4

Cannula infusione curva da Ø 0.80 mm, foro inferiore Ø 0.50 mm, manico in titanio
Irrigating handpiece, Ø 21G, curved tip, 25G underside port, titanium handle

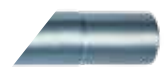
J2233.5

Cannula aspirazione curva, punta ellittica, sabbata, foro superiore Ø 0.35 mm, manico in titanio
Aspirating handpiece, curved, elliptical section tip, sandblasted tip, 28G top port, titanium handle


J2233.6

Cannula infusione curva, punta ellittica, foro inferiore Ø 0.50 mm, manico in titanio
Irrigating handpiece, curved, elliptical section tip, 25G underside port, titanium handle


J2233.10

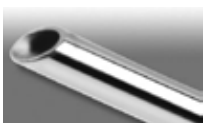Cannula aspirazione, curva da Ø 0.80 mm, foro superiore Ø 0.35 mm, manico in titanio
Aspirating handpiece, Ø 21G, curved tip, 28G top port, titanium handle

J2233.11

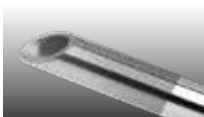Cannula infusione curva da Ø 0.80 mm, apertura in punta flautata Ø 0.60 mm, manico in titanio
Irrigating handpiece, Ø 21G, curved tip, 23G fluted front opening, titanium handle

J2233.12

Cannula infusione curva da Ø 0.80 mm, punta flautata aperta e sabbata da Ø 0.60 mm, manico in titanio
Irrigating handpiece, Ø 21G, curved sandblasted tip, 23G fluted front opening, titanium handle

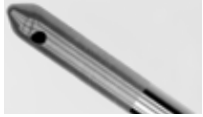
23G I/A BIMANUAL ULTRA THIN CANNULAE

IRRIGATING

ASPIRATING


J2233.80

Cannula aspirazione 23G, **"thin-wall"** foro aspirazione **0.35 mm**, **sabbiata**
23G aspiration handpiece, **"thin-wall"**, **sandblasted**, titanium handle, **0.35 mm** port


J2233.81

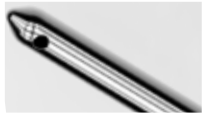Manipolo per infusione 23G, **"thin-wall"**, fori laterali da **0.4 mm**
23G infusion handpiece, **"thin-wall"**, titanium handle, two **0.4 mm**. Side ports


J2233.8

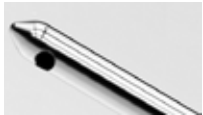Cannula aspirazione 23G, **"thin-wall"**, foro aspirazione **0.35 mm**, **liscia**
23G aspiration handpiece, **"thin-wall"**, **smooth**, titanium handle, **0.35 mm** port

J2233.82

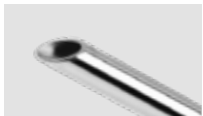Manipolo infusione, 23G, **"thin-wall"**, foro inferiore **0.4 mm**
23G infusion handpiece, **"thin-wall"**, titanium handle, **0.4 mm** lower port

J2233.83

Manipolo infusione, 23G, **"thin-wall"**, liscio, **apertura frontale diagonale**
23G infusion handpiece, **"thin-wall"**, smooth, **front diagonal opening**, titanium handle

I/A BIMANUAL ULTRA THIN 23G DISPOSABLE CANNULAE 5 per box sterile

JD96624

Manipolo per infusione 23G, **"thin-wall"**, punta curva, fori laterali da 0.4 mm
23G infusion handpiece, **"thin-wall"**, two 0.4 mm. Sideports

JD96613

Cannula aspirazione 23G, **"thin-wall"**, punta curva, foro aspirazione 0.3 mm, **sabbiata**
23G aspiration handpiece, **"thin-wall"**, sandblasted, 0.3 mm port (curved)

JD96629

Cannula aspirazione 23G, **"thin-wall"**, punta curva, foro aspirazione 0.3 mm, **liscia**
23G aspiration handpiece, **"thin-wall"**, 0.3 mm port (curved)

I/A CANNULAE COAXIAL

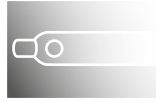
J2239.30 Manico ergonomico per cannule I/A Ergonomic handle for I/A cannulas

da usare senza lo sleeve in silicone
to be used without the silicone sleeve

J2239.31 ★ BEST SELLER

I/A cannula, **diritta/straight**

J2239.33

I/A cannula, **angolo di 90°/angled 90°**

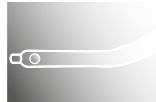
J2239.32 ★ BEST SELLER

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J2239.35

I/A cannula, **curva/curved**

J2239.34

I/A cannula, punta 150° **curva uncino ore 12**
tip 150° **hook-shaped**

I/A CANNULAE COAXIAL WITH SILICONE SLEEVE

sistema composto da manipo in titanio, tips angolata 45° e sleeve in silicone / system composed by titanium handle, cannula angled 45° and silicone sleeve

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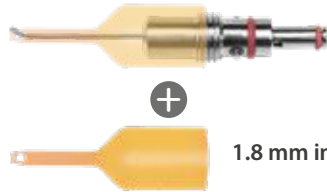
MICS I/A cannula angled 45°, for **1.8 mm incisions**
to be used with silicone sleeve PA603904

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SLEEVE (disposable/sterile)
consisting of 10 infusion sleeves. Size **23G**.

To be used with J2239.38 cannula, 1.8 mm incision

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MICS I/A cannula angled 45°, for **2.2 mm incisions**
to be used with silicone sleeve PA603903

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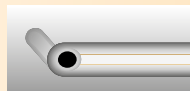
SLEEVE (disposable/sterile)
consisting of 10 infusion sleeves. Size **21G**.

To be used with J2239.36 cannula, 2.2 mm incision

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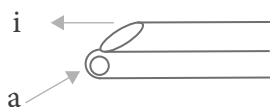
23G irrigation cannula, fluted-front opening, through needle hub
23G aspiration cannula with 28 G port, through silicone tubing hub

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25 G hydrodissection cannula,
12 mm angle to tip,
curved and flat, front opening
designed by Buratto M.D.

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25 G hydrodissection cannula,
flat, blunt tip, end opening,
12 mm angle to tip

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27 G hydrodissection cannula,
flat, blunt tip, end opening,
7 mm angle to tip

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Nucleus hydrorotation cannula,
27 G, angled, hook-shaped tip, end opening
designed by Caporossi M.D.

J2252.1

Cannula irrigante per nucleo, Ø 1.20 mm,
con ansa poco curva, a 3 fori frontali

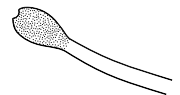
18 G nucleus irrigating cannula,
slightly curved loop, with 3 front openings
designed by Pearce M.D.

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25 G capsule polisher cannula,
curved, sandblasted, olive tip, end opening
designed by Kratz M.D.

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23 G cortex aspirating cannula,
curved, long, end opening
designed by Charleux M.D.

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23 G cortex aspirating cannula,
curved, short, end opening
designed by Charleux M.D.

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25G cannula, angled, blunt hook, 1 mm, for 12 o'clock position in, front port

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25 G capsule polisher cannula, angled, 2 mm angle to tip, sandblasted tip, end opening

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21G Scraper cannula, sandblasted tips for a delicate removal of cortical residues, front port

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25G viscoelastic aspirating cannula, angled, 12 mm angle to tip, open end

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23G viscoelastic aspirating cannula, angled, 7 mm angle to tip, open end

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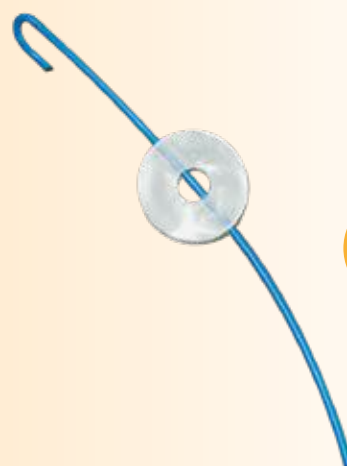
23G (ø 0.6) spatulated anterior chamber maintainer, with grooves to improve steadiness. Lower port grants improved lenticle fluids dynamics in DMEK procedures. Lower hole infusion reduces dramatically unwanted lenticle motion which is definitely present with a standard infusion. DESIGNED FOR THE DMEK TECHNIQUE "PULL-THROUGH" by Massimo Busin M.D.,Ph.D.

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Anterior chamber maintainer, **20G** self-retaining, atraumatic insertion, titanium

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Anterior chamber maintainer, **23G** self-retaining, atraumatic insertion, titanium

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Anterior chamber maintainer, **23G** self-retaining, atraumatic insertion, titanium. **SHORT** (2mm length tip)

MANTENITORE C.A./A.C. MAINTAINER 5 per box sterile

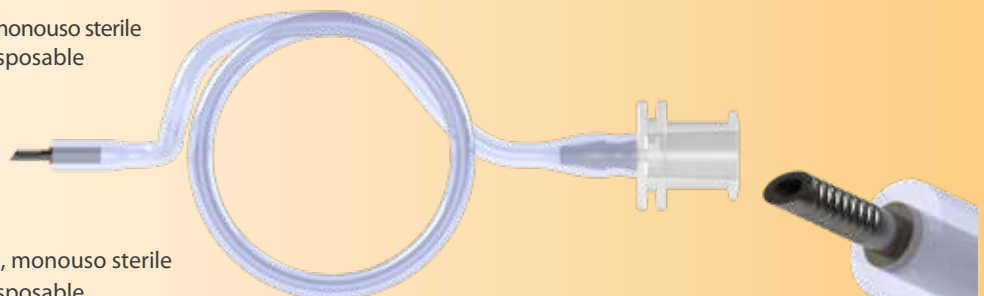
JD93768 A/C maintainer

Mantenitore camera anteriore da **20G**, monouso sterile
Anterior chamber maintainer, **20G**, disposable


JD93769 A/C maintainer

Mantenitore camera anteriore da **23G**, monouso sterile
Anterior chamber maintainer, **23G**, disposable

PRE-OPERATIVE MARKERS _ PENDULUM SWINGING SYSTEM

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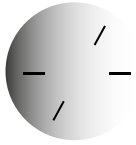
Axial marker, for horizontal/toric axis marking, allows **four pre-operative marking imprints**

- With the horizontal axis marker fixed, the toric axis marker can be rotated and fixed at the requested angle with a 5° step mechanism, thus providing in one step both, horizontal and toric axis marking
- Pendulum swinging system assures precise horizontal axis
- One-step marking of 2 axis: horizontal and toric

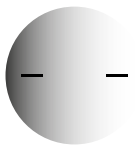The pendulum systems device require a correct position of the patient sitting at the slit lamp
designed by Elies M.D.

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

Axial marker with eye **centering device**, horizontal axis performed by marking epithelium imprint.

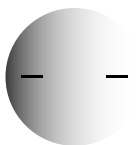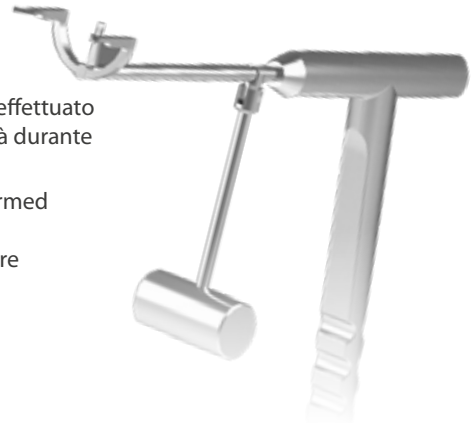Curve Shaped for better visibility during the marking procedure
(based on Elies M.D. model)
designed by Fazio M.D.

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Axial marker, for **horizontal axis (0-180° limbus marking)**, for patient seated at the slit-lamp. Hinge to grant the pendulum to work perpendicular to the ground in any holding position
designed by Elies M.D.

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Semi-circle marker **0-90°, 0-90°**, for repositioning of toric lenses
designed by Soler M.D.

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
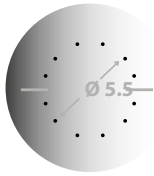
Toric axial marker and capsulorrhexis gauge Ø 5.5 mm.
Recommended use with Mendez ring J2294.5 and new Ligabue gauge ring J2294.56
It allows to fix the toric axis and leaves a precise marking spot simultaneously.
Useful to perform the recommended 5.5 mm capsulorrhexis for toric IOL implantation designed by E. Ligabue M.D. VIDEO SU WWW.JANACH.IT

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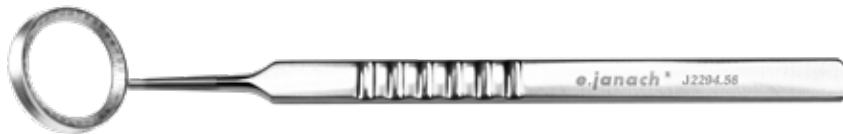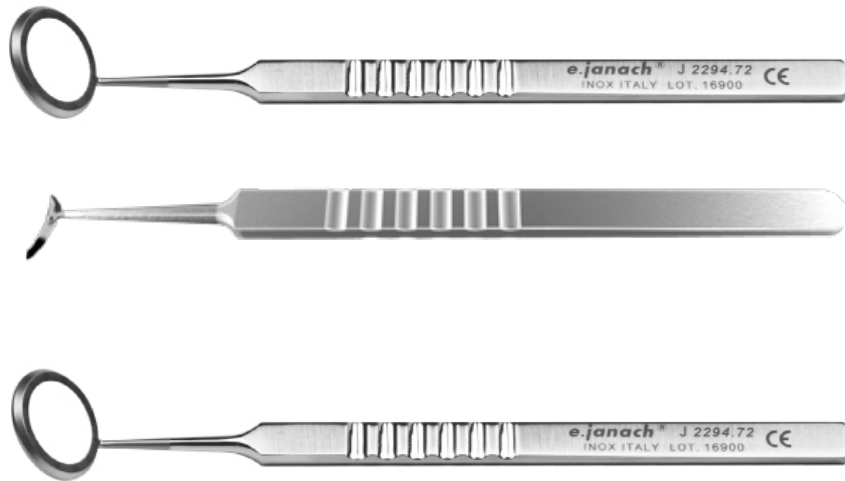
Gauge ring for intra-operative toric axis marking. Inner Ø 11.5 mm. The edge of the ring has an atraumatic and ergonomically section designed to reduce stress on ocular tissues during the application.

The same geometry allows the ring to gently embrace the bulb, improving the stability of the ring and allowing a more secure and accurate marking

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Axial marker with thinner ray. It leaves indentation into epithelium

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Main Features:

- . ring marker with marks each 5°
- . horizontal axial marker can be also used without the marking pens, it leaves two soft marking imprints on the epithelium

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Cannula for I/A coaxial handle, tips 0°. For Alcon® "ultra sleeve" (2.2mm incisions). Can be used also for Janach sleeve PA603903

Suggested for soft nucleus aspiration, no ultrasound needed

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SLEEVE (disposable/sterile) consisting of 10 infusion sleeves. Size **21G**.

To be used with J2239.36 and J2239.41 cannula, 2.2 mm incision

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The new splitter allows a perfect fragmentation of the small quadrants created with new specific software for nucleus fragmentation. The micro-splitter gently penetrates into each single section thus simplifying the fragmentation process.

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Nucleus splitter for femto "MATRIX" technique. **Left-handed** designed by P. Crozafon® M.D.

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25G (Ø 0.5 mm) hydrodissection cannula, specific for the use with **femtosecond laser**, it removes the residual cortex easily as a result of removing the nucleus fragmented with the laser

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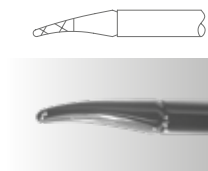
J3167.1

Pinza **coassiale** 23G per rimozione lenticolo **corneale/stromale**, **procedura Smile®**.
Coaxial forceps 23G for lenticle removal, in **Smile® procedure**. **Slightly curved shaft, short, with serration cross hatch**

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This surgical forceps has been specifically designed to facilitate the insertion of corneal lenticules in all the types of corneal intrastromal implantation surgery. Particularly thanks to the cross-action of its branches and to the flat grasping tips it allows to insert the lenticule through small incisions (as small as 3.5 mm) and to spread out and unfold the implanted tissues within the stromal pocket in one single maneuver. Particularly useful for the SLAK procedures (Stromal Lenticule Addition Keratoplasty) in corneal ectasia, can be employed in any kind of intrastromal tissue augmentation procedures
designed by L. Mastropasqua M.D. Ph.D / M. Nubile M.D. Ph.D

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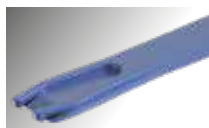
Spatula for delamination of intrastromal layers in refractive and corneal procedures. Highly suggested to delaminate the two intrastromal layers in **Smile® procedure**. Thin and slightly bevelled edges allow a delicate cleavage on the stromal tissue
designed by L. Mastropasqua M.D. Ph.D / M. Nubile M.D.

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Forceps for stromal lenticle removal in **Smile® procedure**, titanium designed by M. Surace M.D.

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Forceps for lenticle removal in **Relex® procedure**, stainless steel (MILLY smile) designed by Morselli M.D. VIDEO SU WWW.JANACH.IT

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Double ended instrument for **Smile® procedure**. Spatula for detachment and delamination of lenticle, hook for opening of the incision designed by M. Surace M.D.

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Double ended instrument for **Smile® procedure**. DISK spatula for detachment and delamination of lenticle, hook for opening of the incision. designed by M. Surace M.D.

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Double ended instrument for **Smile® procedure**

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Intrastromal pocket dissector, disk-shaped 1.5 mm, slightly curved, to open the intrastromal pocket in **Smile® procedure** designed by L. Mastropasqua M.D. Ph.D.

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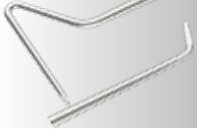
Double-ended instrument to unstick and lift the corneal flap in LASIK procedures. The delicate hook works at the flap edges inside the groove. The "C" shaped tapered loop, once positioned below the flap, lifts it with a gentle push on the horizontal plane (from the hinge to the exit point)

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Double-ended instrument tapered tips to unstick and lift the corneal flap in LASIK procedures. The angled tapered tip works at the flap edges inside the groove. The linear tapered tip, once positioned below the flap, lifts it under a gentle push on the horizontal plane (from the hinge to the point of exit)

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Double-ended instrument to unstick and lift the corneal flap in LASIK procedures. The delicate hook works at the flap edges inside the groove. The lanced shaped tip, once positioned below the flap, lifts it under a gentle push on the horizontal plane (from the hinge to the point of exit)

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Double-ended instrument to unstick and lift the corneal flap in LASIK procedures. The delicate angled tip works at the flap edges inside the groove. The tapered straight tip, once positioned below the flap, lifts it under a gentle push on the horizontal plane (from the hinge to the point of exit)

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Double-ended instrument to lift the corneal flap in LASIK procedures. The delicate tapered tips have different lengths and angles. The shortest tip, less angled, is used for temporal access, while the longest tip, with wider angle, is used for nasal access. Both tips, once positioned below the flap, lift it under a gentle push on the horizontal plane (from the hinge to the point of exit)

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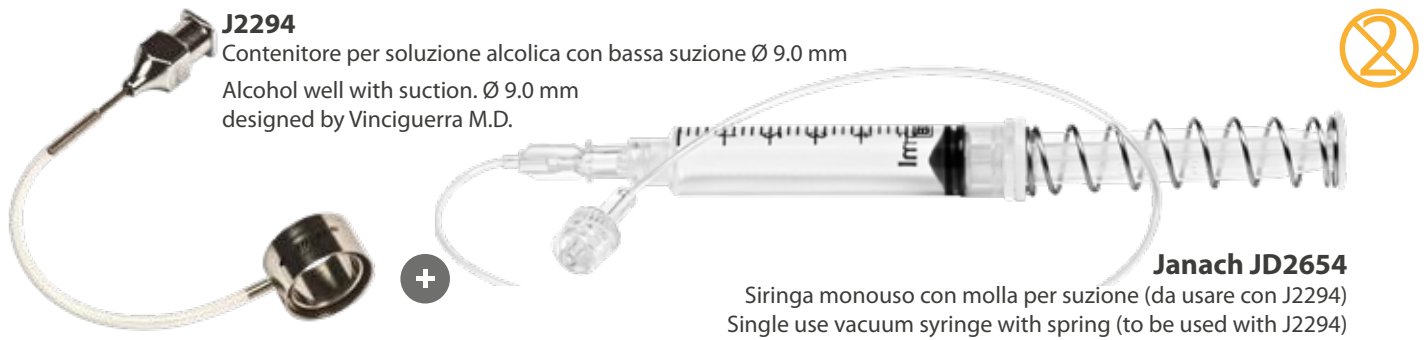
J2294.91S Corneal Marker (Provides a circle mark on the cornea)

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A cross-hair helps surgeons to put a corneal mark on a target position precisely

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Kamra® Inlay Inserting Forceps has a cross-hair on the bottom side of the forceps to help surgeons inserting Kamra® Inlay easily and smoothly into a target position. Dr. Tomita's unique design curved forceps make minimum stress on the pocket incision and fasten patients' visual recovery

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Double marker for limbus and corneal flaps in LASIK procedure.

It is extremely important for surgeons to make a pocket incision outside of LASIK flap

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Inlay Positioner for repositioning Kamra Inlay has friction on its bottom side.

Once Kamra® Inlay is inserted into a pocket, it is difficult for surgeons to reposition the Kamra® Inlay. This friction helps the surgeons to move the Kamra Inlay into a target position smoothly and precisely with minimum invasion

BOTTOM SIDE

J2180.91A: Hook for Kamra® Inlay implantation

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This hook is to open the pocket sidecut and to lift the incision while inserting Dissector, Kamra® Inlay Inserting Forceps and Positioner

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Intrastromal pocket dissector. The unique design, curvature and thickness of this dissector allows comfortable dissection. Of the pocket without stressing the incision

ICL COAXIAL FORCEPS designed by Aus Der Au D.M.

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ICL™ loading coaxial forceps, atraumatic angled tips. TIPS OPENING PARALLEL TO THE CLOSING MECHANISM OF THE SQUEEZE HANDLE

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ICL™ loading coaxial forceps, atraumatic angled tips. TIPS OPENING PERPENDICULAR TO THE CLOSING MECHANISM OF THE SQUEEZE HANDLE

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ICL COAXIAL FORCEPS designed by M. Tomita M.D.

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Spatula, slightly curved, flat delicate tip for positioning ICL™ lens

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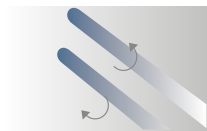
ICL™ lens manipulator, flat atraumatic disc-shaped tip

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Foldable lens cartridge loading forceps 11 mm angle to tip cylindrical arms

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ICL™ angled loading forceps, delicate tips. 13 mm long thin blades

IRIS AND SCLERAL FIXATED IOL INSTRUMENTS

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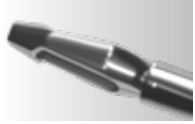
23G coaxial holder, slightly curved shaft, for scleral fixation lenses with T-shape/anchor loops.

The design of the forceps allows you to gently embrace the loops of the lens (IOL) at the base, managing the insertion through the scleral incisions. The implantation of the IOL results in safety and ensures repeatability designed by A. Pioppo M.D.

J3009.20L

Holder coassiale 20G per IOL fachiche

20G coaxial holder for phakic IOL designed by L. Buratto M.D.

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Coaxial holder for enclavation of foldable iris-fixated phakic IOLs. Designed by L. Buratto M.D.

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Double hook for enclavation of iris-fixated phakic IOLs

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Implantation forceps for foldable iris-fixated IOL, long tip designed by F. Carones M.D.

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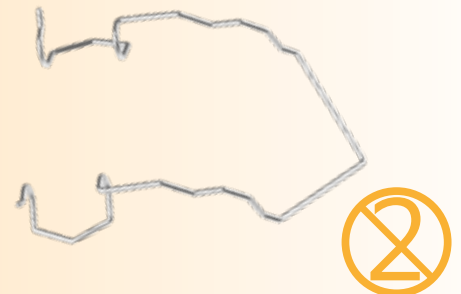
wire speculum disposable 15 mm blade with large frame. Recommended for diagnostic and surgical procedures. The wide frame allows easy placement and removal

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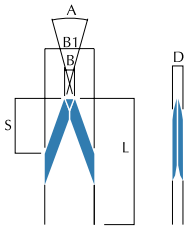
Lama retta a lancia tri-facet, double bevel, manico a baionetta in titanio


single cutting depth

Tri-facet lancet shaped blade, double bevel, titanium retractable handle

Order No	Cutting depth	B-B1	L	D	A
J6800	0.60	0.40-1.00	2.00	0.20	40°
J6801	0.65	0.40-1.00	2.00	0.20	40°
J6804	0.55	0.40-1.00	2.00	0.20	40°

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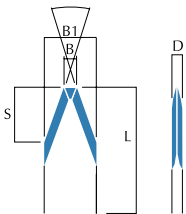
three-preselectable cutting depths

Tri-facet lancet shaped blade, double bevel, 3 preset cutting depth, titanium retractable handle

Order No	Cutting depth	B-B1	L	D	A
J6807	0.50-0.55-0.60	0.40-1.00	3.00	0.10	20°

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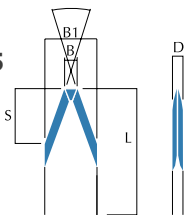Lama retta tri-facet, double bevel, manico in titanio con micrometro


three-preselectable cutting depths

Double bevel, tri-facet straight blade, titanium handle with micrometer

Order No	S	B	L	D	A
ME425	0.2	0.8	3.5	0.1	15°

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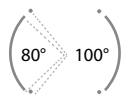Astigmatism markers

Arcuate incision marker, Ø 11 mm markings at 80° - 100° (To be used with J2295.5)

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Arcuate incision marker, markings at 0° - 35° - 45° - 55°

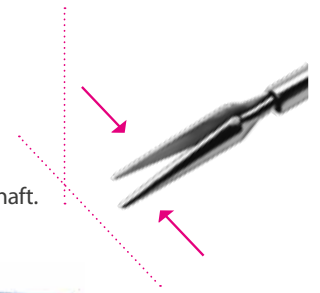
23G ANTERIOR CHAMBER SURGERY AND SCLERAL IOLS FIXATION COAXIAL TOOLS

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
23G multipurpose microscissors, for Anterior Chamber surgery, **HORIZONTAL CUT**, curved shaft. Also recommended for adherent synechiae.

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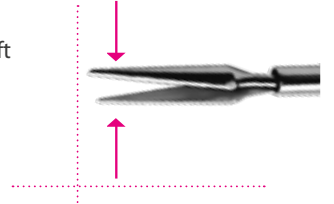23G multipurpose microscissors, for Anterior Chamber surgery, **VERTICAL CUT**, curved shaft. Useful for capsulorrhexis correction and IRIS sphincterotomy

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Useful in Anterior Chamber surgery. It can be used as a holder when the IOL is removed as well as needle holder in IRIS suturing

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23G Coaxial micro forceps **Mc Pherson**. The forceps is suggested to perform Anterior Chamber surgery.

Useful to manage IOL into the Anterior Chamber and fix the suture of the IRIS designed By L. Buratto M.D.

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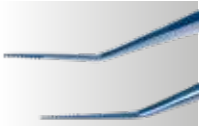
Cutting scissors for removal IOL, angled, 6 mm long, delicate blades, titanium. Serrated lower blade to hold the IOL while cutting

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Holder forceps with 5 mm serrated tips, 13mm angled to tip, titanium. Useful to better fix the optical' lense when is removed

J3871

Microforbice coassiale per taglio anse IOL, 20G, in titanio

Coaxial Haptic Cutter, 20G, titanium handle

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IOL removal 19G coaxial microscissors (to be used with holder J3873L)
To be used for soft silicon IOLs only

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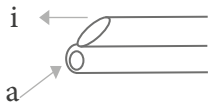
20G coaxial holder for IOL removal (to be used with IOL removal microscissors J3873L)

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Ansa zigrinata, da 5 mm
Serrated lens loop, 5 mm wide

Snellen J2592

Ansa zigrinata, da 4 mm
Serrated lens loop, 4 mm wide

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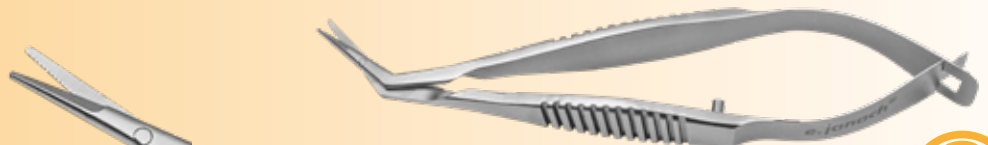
Descemet membrane punch, \varnothing 1 mm, by Kelly

J2865

Punch corneo-sclerale, retto, \varnothing 1.5 mm

Corneal-scleral punch, straight, \varnothing 1.5 mm

J2867

\varnothing 1.7 mm

\varnothing 1.7 mm

J2869

Punch da 20G, taglio da 0.75 mm per trabeculectomia

20G trabeculectomy bite punch, 0.75 mm tip For small sclerectomy

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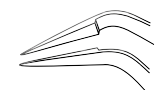
Stripping forceps for the Schlemm canal, **curved tip**

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Forceps for stripping of Schlemm canal

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Clamp to protect conjunctival edge during antimetabolite application

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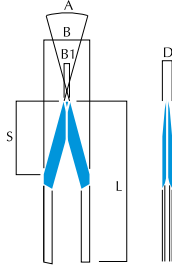
Caliper, straight, 20 mm spread, graduated scale with 0.5 mm steps

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
Tri-facet lancet-shaped blade, double bevel, titanium handle with pre-selectable cutting depth

Order No	B	B1	L	D	S	A	Pre-selectable cutting depth
J6412	1.00	0.30	3.50	0.20	1.60	25°	0.15 0.20 0.25 0.30 0.37 0.50 4.00

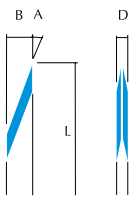J6204DA

Lama retta da 20°, double bevel

Double bevel, 20° single blade cutting edge

Order No	B	L	D	A
J6204 DA	0.50	3.50	0.20	20°

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Deep-sclerectomy scissors, straight, 6 mm long, delicate blades, blunt tip on lower blade

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Double scleral marker for deep sclerectomy in glaucoma surgery. Flap dimensions: 4x5 mm and 5x4 mm for myopia

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Superior muscle hook, curved, **with hole on distal point**

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STERILIZATION BOX, aluminium, **300X300X110, PROTECTED VALVE**, with blue lid and handles

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STERILIZATION BOX **285 X 185 X 65 mm.**
Fits 1 tray J4564 and uses filter J4579.40
(both to be purchased separately)

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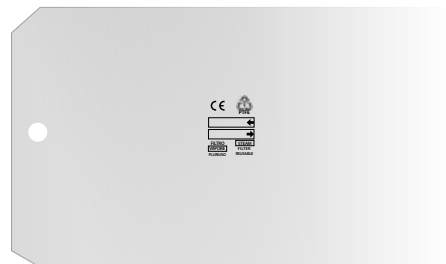
STERILIZATION BOX **285 X 185 X 95 mm.**
Fits 2 trays J4564 and uses filters J4579.40
(both to be purchased separately)

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PTFE FILTER, 240 X 145 mm. For use with
sterilization boxes J4577 and J4578.
10 filters per box

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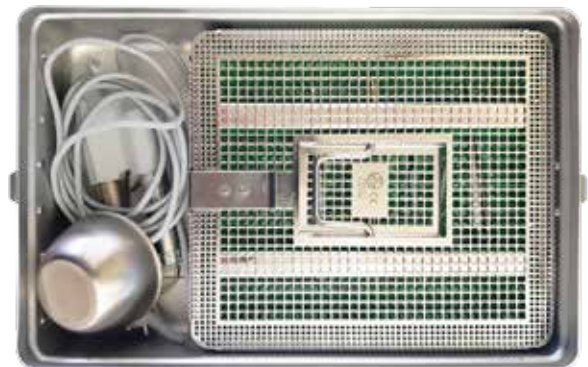
PERFORATED BASKET 270 X 170 X 35 mm,
with silicone mat and silicone stopper.
Fits boxes J4577 and J4578

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PERFORATED BASKET with handles
200 X 170 X 35 mm,
with silicone mat and silicone stopper

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Combination of:
STERILIZATION BOX J4577 +
PERFORATED BASKET J4564.1

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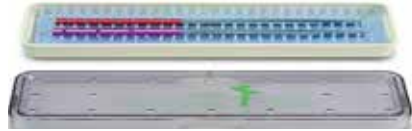
PSU tray with stop, for 6 instruments,
20.3 x 11.4 x 1.5 cm (internal dimensions)

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PSU tray with stop, for 12 instruments,
20.3 x 11.4 x 3.0 cm (internal dimensions)

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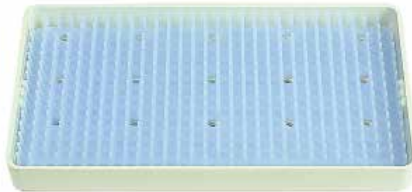
PSU sterilization tray, with silicone mat, outer dimensions: 18.1 x 4.9 x 1.8 cm
Recommended for 2 retinal instruments

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PSU sterilization tray, with silicone mat, fits 5-7 instruments, outer dimensions: 19.9 x 11 x 2.5 cm

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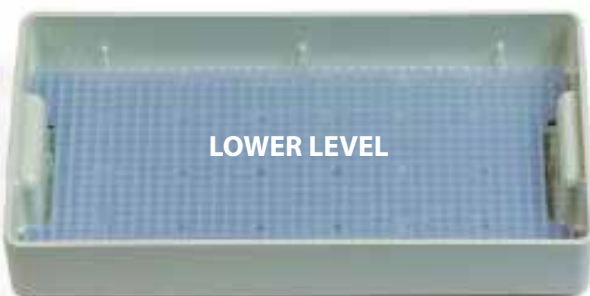
PSU sterilization tray, with silicone mat and metal clips, fits 10 instruments, outer dimensions: 28.3 x 17.5 x 2.5 cm

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PSU sterilization tray, with silicone mat and metal clips, two tiered, fits 20 instruments, outer dimensions: 28.3 x 17.5 x 4.5 cm

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Designed for the quick and efficient management of fluids during LASIK procedures.

Ultra-smooth, biocompatible PVA sponge. Ultra-fast wicking, lint and fibre free. Supplied sterile, single use only, declared 5 year shelf life

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
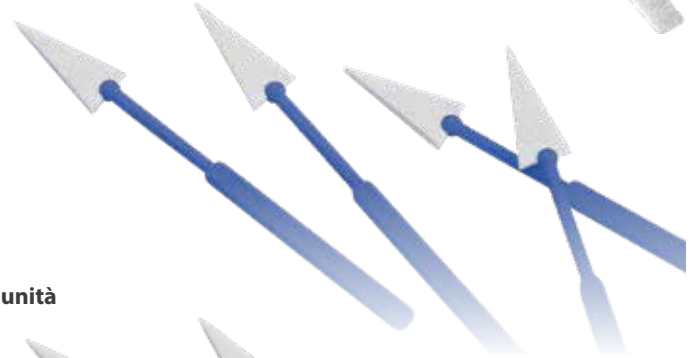
PVA spears with handle. **box of 200 items 20 pack of 10 sterile units**

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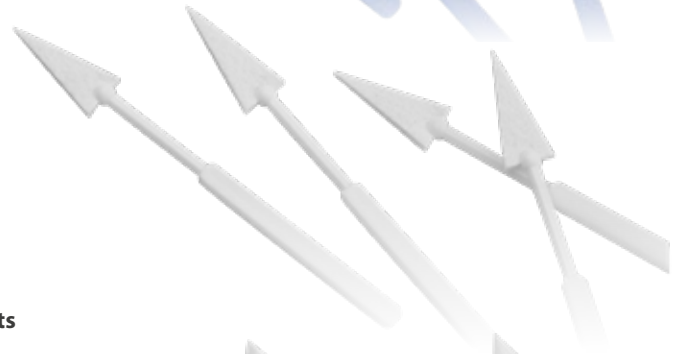Triangular sponges with handle. **box of 200 items 20 pack of 10 sterile units**

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Viscotex sponges, 5x66 mm. **box of 400 items 20 pack of 20 sterile units**

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TSL 44, detergent and disinfectant solution for ophthalmic contact items

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Multi-enzymatic detergent system. Specific for the preliminary removal of blood and organic contaminants from devices to be later disinfected or sterilized. Suitable for manual immersion washing, for washing and ultrasounds machines

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FLUOR EYES

grasp tabs between thumbs & index fingers

gently pull tabs apart

remove strip taking care to touch only the white portion of the strip

FLUORESCEINA

STRISCE OFTALMICHE / OPHTHALMIC STRIPS

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It simplifies phacoemulsification and surgical interventions in which the pharmacological dilation of the pupil is not sufficient.

- Semicircular 300° ring with opening at 60°, ring in single piece
- Injector not needed
- Easy insertion and removal
- The ring can be inserted through incisions less than 2 mm
- Easy to fit into pupils less than 4.5 mm diameter
- Excellent visibility
- Single use device / cannot be resterilized

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AJL BBG 0.025% is an ophthalmic sterile, non-pyrogenic, isotonic solution, ready for use in vitreoretinal surgery. The BBG solution is capable of effectively staining

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It is an ophthalmic sterile trypan blue solution, intraocular and biocompatible used in ophthalmic surgery of the anterior segment. This ophthalmic dye is used as an adjuvant during capsulorhexis and minimises the risk of laceration in eyes with mature cataracts or with small pupils

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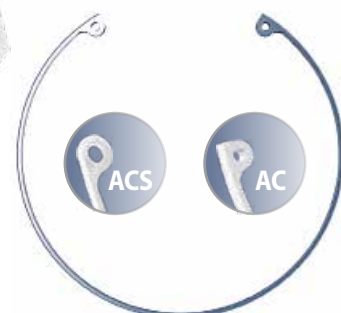
Surgical solution to gently expand insufficiently dilated pupils during ophthalmic surgery and sustain visibility throughout the procedure

- Unimpeded view during complicated small pupil surgeries
- Four pockets that gently cradle the ocular tissue without pinching or clamping.
- Once removed, the iris returns to its normal shape and function
- Also used to manage Floppy Iris Syndrome (IFIS)
- Easy removal without potential tearing

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LONG TERM OCCLUSION

The distinctive low-profile cap combined with a softer, flexible silicone maximizes patient comfort. Now available with two inserter options. SOFT PLUG® Silicone Punctum Plugs are removable with forceps per instruction for use

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LONG TERM OCCLUSION WITH PARTIAL DRAINAGE

The distinctive low-profile cap combined with a softer, flexible silicone maximizes patient comfort. The inserter minimizes potential for accidental release of the plug. Plug is designed with a center channel allowing for partial tear flow. SOFT PLUG® Flow Control Plugs are removable with forceps (per instruction for use)

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© Copyright: